

A Novel Blood-based Assay that Predicts Clinical Response to TNFi or JAKi in Patients with Rheumatoid Arthritis

Signe Fransen, Katharine Dilger, Kevin Lai, Diana Abdueva, Maggie C. Louie, David Chernoff and Jeffrey R. Curtis

Aqtual, Hayward, CA, USA; Division of Clinical Immunology and Rheumatology, The University of Alabama at Birmingham, Birmingham, AL, USA: For questions or trial information, email Signe.Fransen@aqtual.com


Background

- The heterogeneity of rheumatoid arthritis (RA) poses challenges in achieving optimal clinical outcomes for all patients.
- RA is treated first with conventional synthetic disease-modifying anti-rheumatic drugs (csDMARDs) followed by biologic DMARDs (e.g. tumor necrosis factor- α inhibitors, TNFi) or targeted synthetic DMARDs (janus kinase inhibitors, JAKi), aiming to suppress the immune system's overactive response.
- These treatments have been instrumental in controlling symptoms and slowing disease progression, however not all patients will respond to a specific therapy resulting in patients failing multiple therapies before finding an effective treatment.
- The variability in individual responses to standard therapies raises questions about the underlying factors contributing to this heterogeneity in response and underscores the need for a more targeted approach.
- In this study, we present the development of a non-invasive blood-based classifier to differentiate RA patients likely to respond to TNFi or JAKi therapy from those who will not.


Methods

Overview of Platform

- Aqtual's novel platform provides a non-invasive, real-time assessment of organ- and disease-specific epigenomic and transcriptomic signals.
- Cell-free DNA (cfDNA) was extracted from 1 mL of plasma using a proprietary chromatin capture workflow, followed by library preparation and sequencing.

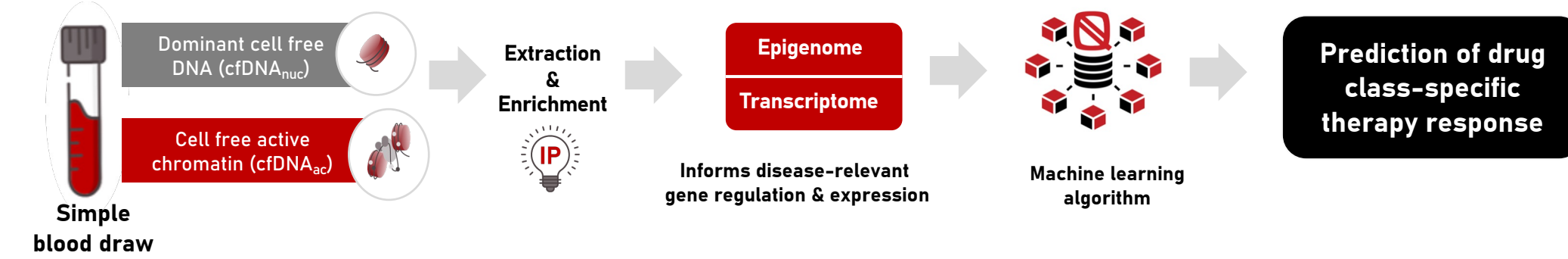


- Response- and therapy- specific signatures were identified using the following machine learning (ML) framework:

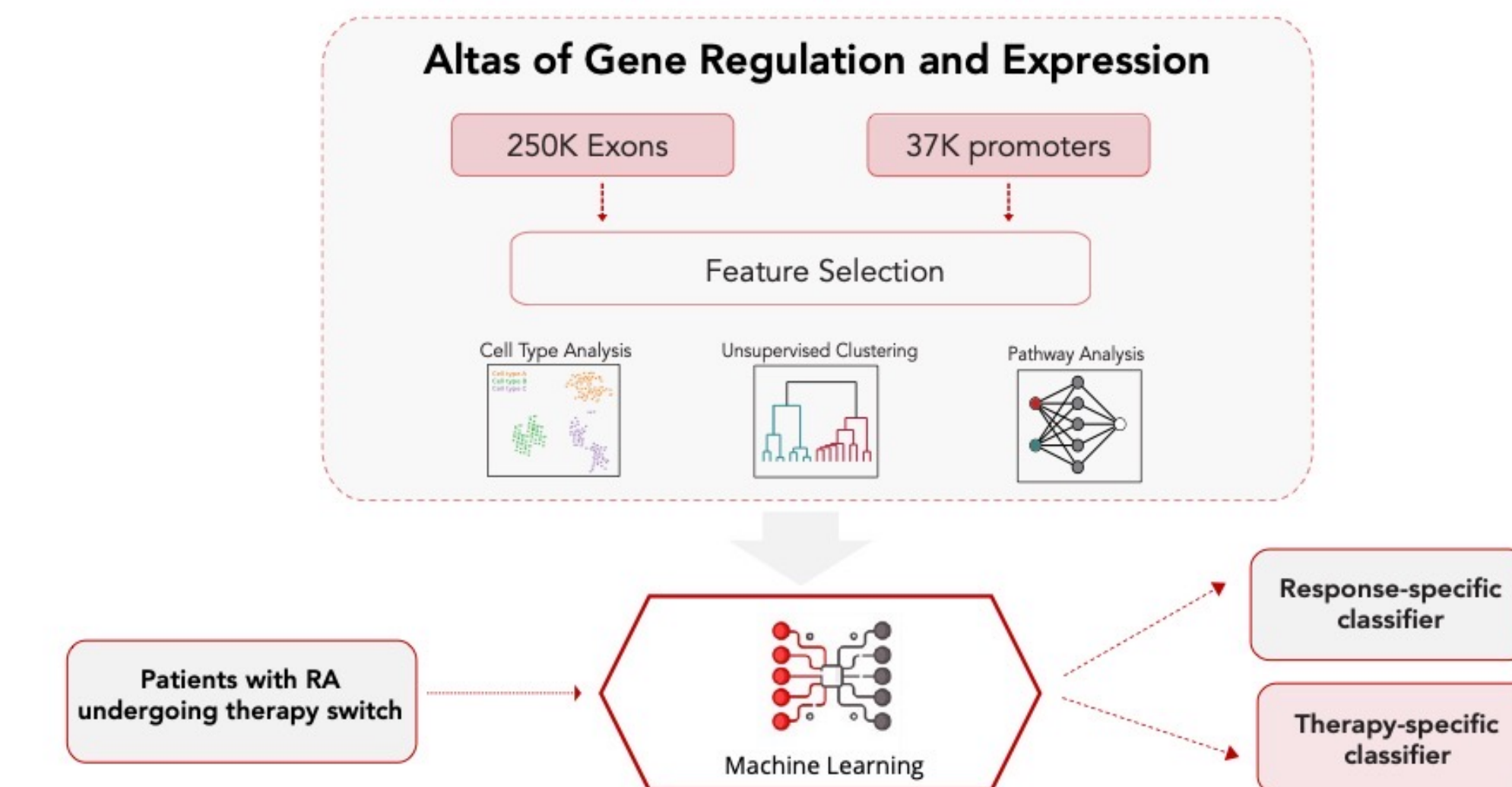

Study Design


Study Objective

- The objective of this study is to evaluate the clinical feasibility of the assay to differentiate therapy response in two drug classes – TNFi and JAKi – in patients with Rheumatoid Arthritis.

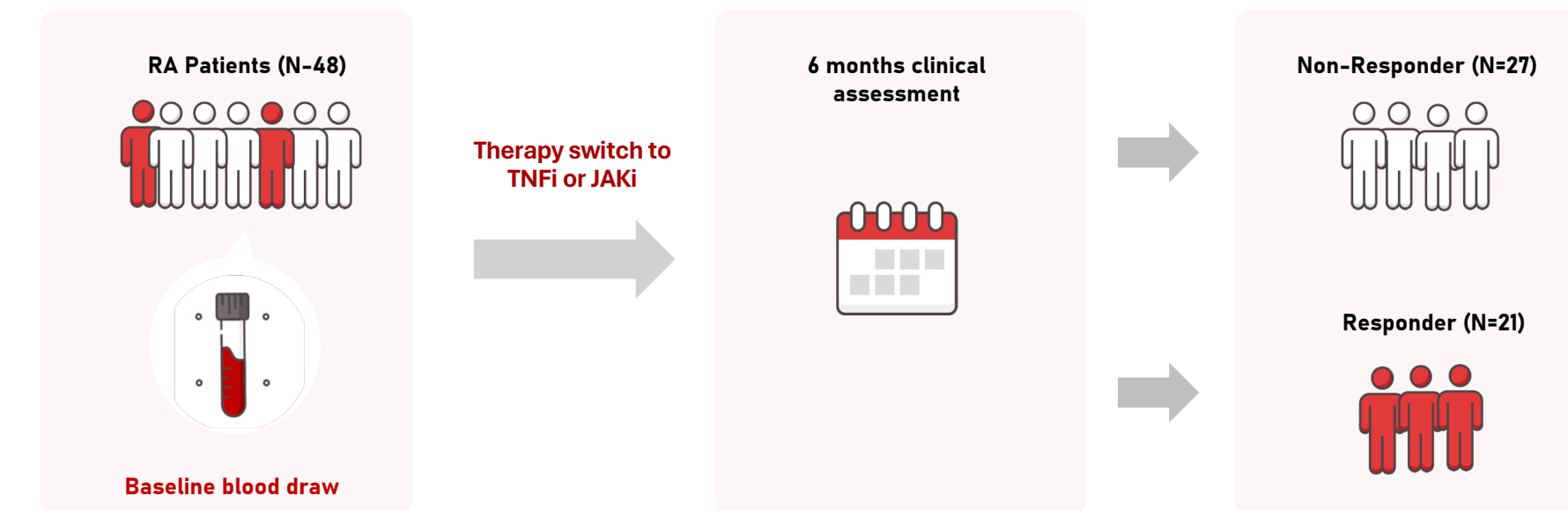

Cohort Description

- A total of 48 RA participants undergoing a switch in therapy to either TNFi (n=19) or JAKi (n=29) were selected from the CorEvitas BIO-100 registry. Sample selection was based on equivalent numbers of responders and non-responder.
- The participants, aged 20–80 years (median=57), consisted of 89.5% females and 10.5% males, with 89.6% identifying as White.
- 21 patients were classified as responders and 27 as non-responders based on their Clinical Disease Activity Index (CDAI) scores collected at baseline and 6 months after therapy initiation.
- The distribution of participants by baseline and follow-up CDAI, and change in CDAI:

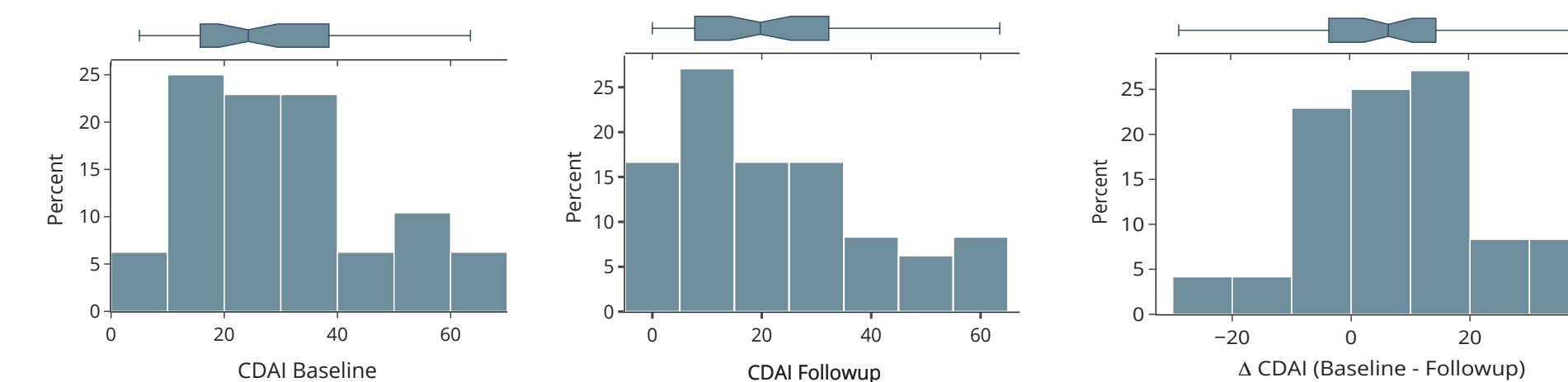


Definition of Response Classes for ML Classifier

Total Cases	TNFi	JAKi
Responders (N)	7	14
Non-responders (N)	12	15

ML Training Cases	TNFi	JAKi
Responders (N)	7	14
Non-responders (N)	6	9


CDAI allows immediate treatment decisions to be based entirely on clinical criteria. CDAI is calculated as: CDAI = SJC + TJC + PGA + EGA

- **Responder:** Disease activity going from high to moderate with a reduction of at least 12 points on CDAI
- **Responder:** Disease activity going from moderate to low with a reduction of at least 6 points on CDAI
- **Non-Responder:** CDAI increased from baseline to follow-up
- Cases with minimal change in CDAI that were excluded in the training of the ML classifier

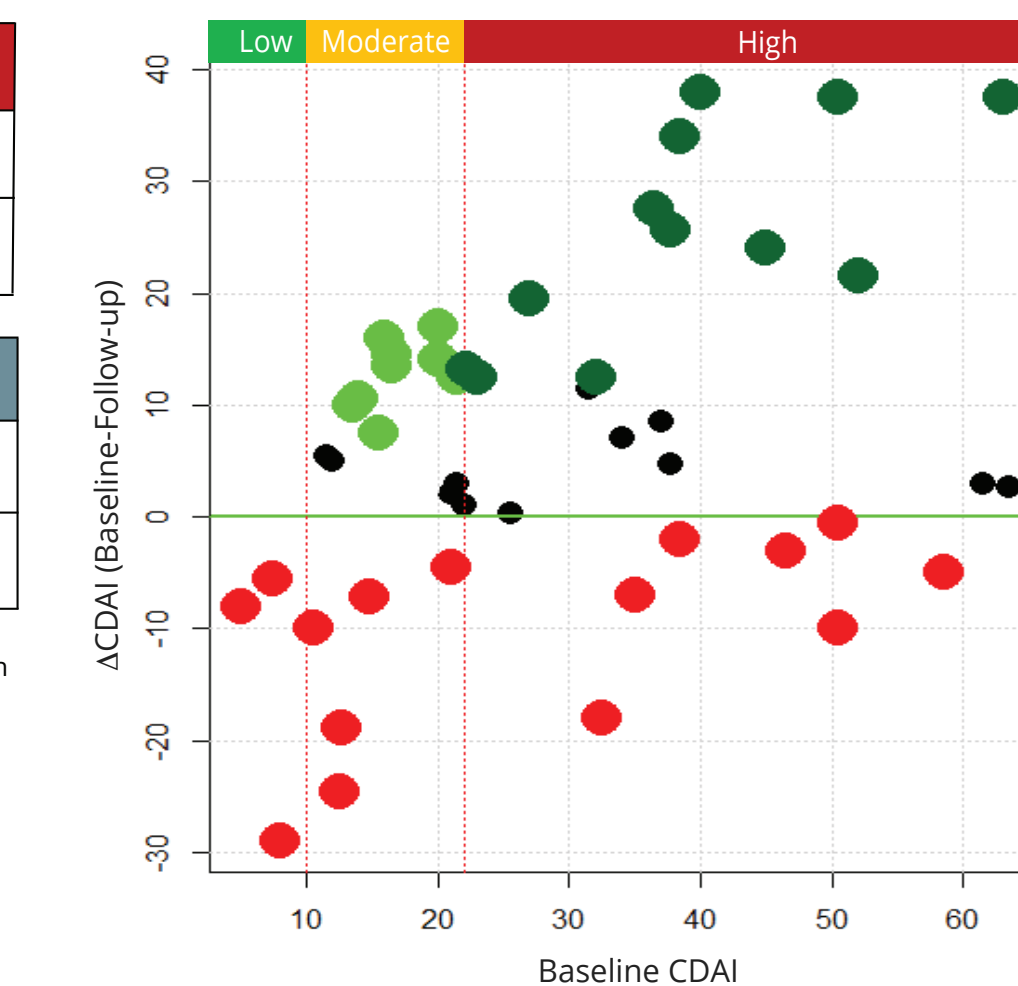

Results


Figure 1. The therapy response classifier in the combined TNFi & JAKi cohort demonstrated a sensitivity of 80% at a specificity of 85%

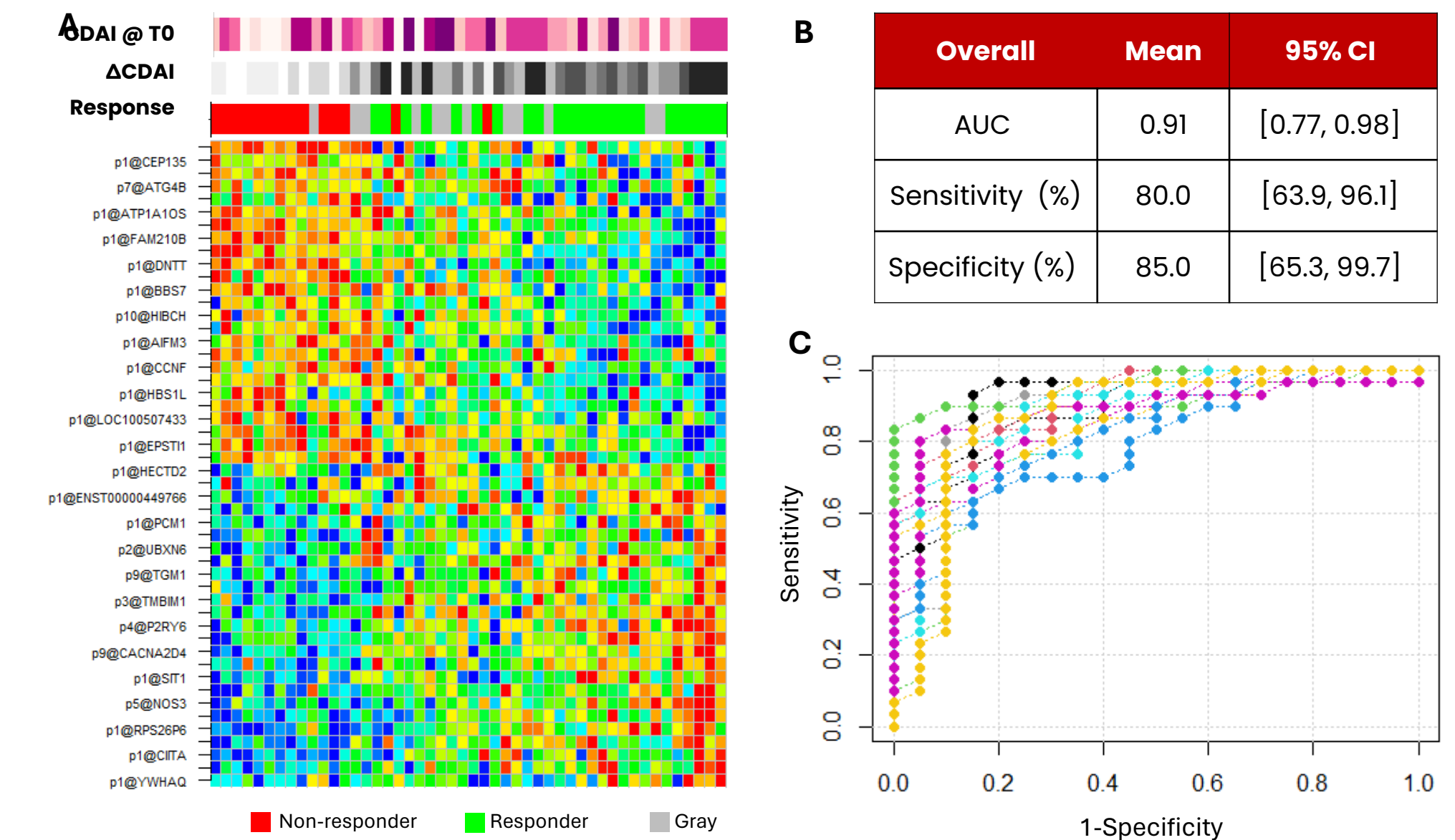

- Change in CDAI score, not baseline CDAI, was associated with treatment response

A total of 57 features with the most discriminatory power were identified, followed by false discovery rate (FDR) correction. **A.** The heatmap displays features identified through hybrid unsupervised clustering during feature selection. Upper tracks denote response status, disease activity, and change in CDAI from baseline to 24 weeks post-therapy initiation. **B.** A machine learning classifier was used to identify relevant variables to construct a predictive model using epigenetic data for patients switching therapies to TNFi and JAKi drug classes. **C.** The ROC curve shows the performance of the classifier, achieving 80% sensitivity and 85% specificity. The workflow incorporated 10,000 classification trees and a 5-repeated 5-fold cross-validation.


Figure 2. Classifier features predicting response are linked to RA pathobiology

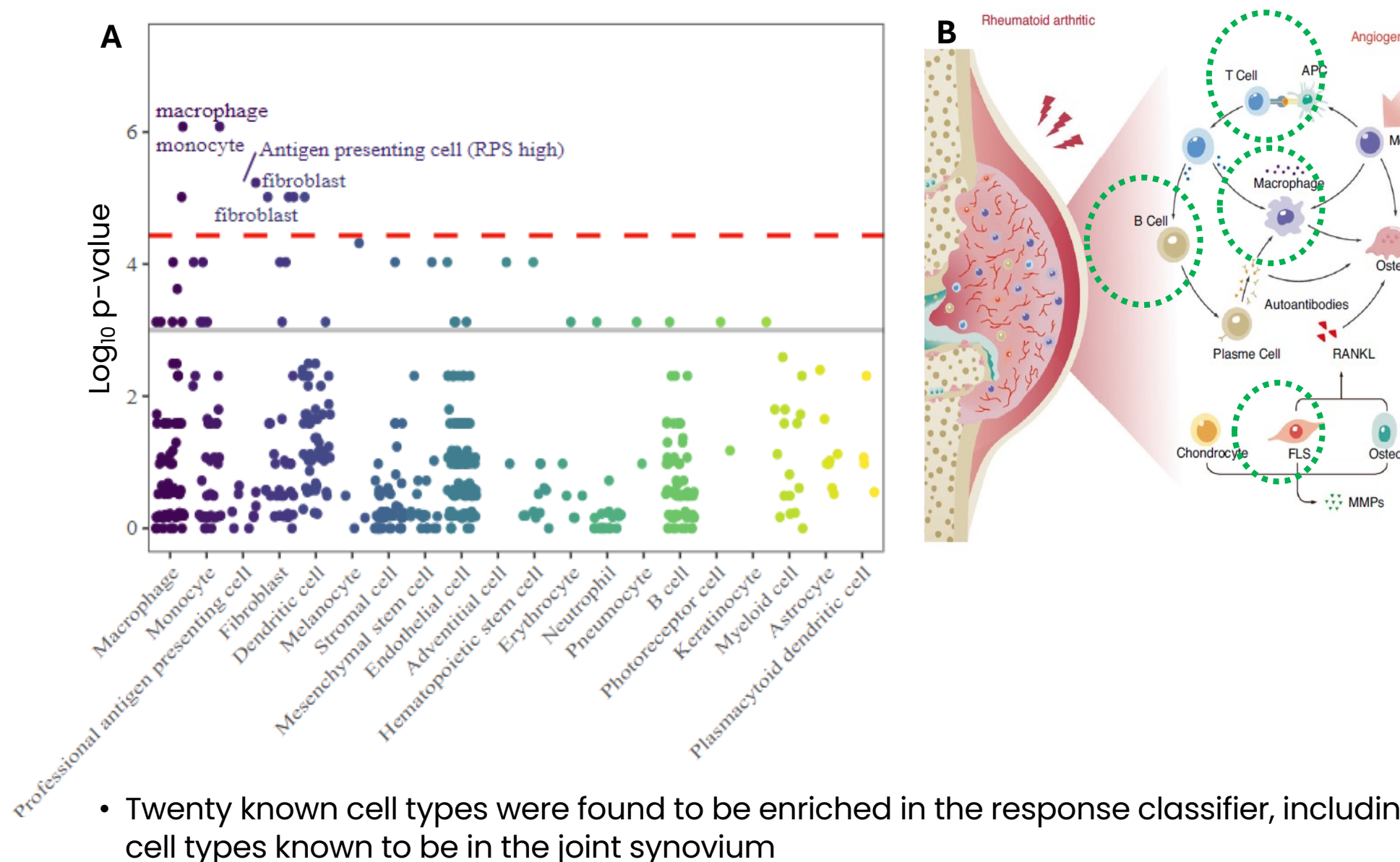

- Twenty known cell types were found to be enriched in the response classifier, including cell types known to be in the joint synovium

Cell-type specific enrichment analysis was performed on feature set mapped to genes using WebCSEA (<https://bioinfo.uth.edu/webcsea/>). **A.** Results show enrichment across top 20 general cell types with raw and Bonferroni-corrected p-values. Bonferroni-corrected significance ($p = 3.69 \times 10^{-3}$) is indicated by a red-dashed line, and nominal significance ($p = 1 \times 10^{-3}$) by a grey-solid line. X-axis represents cell types and y-axis represents the significance level. **B.** Diagram of joint synovium with cell types identified in enrichment analysis shown.

Figure 3. A second ML classifier focused on therapy-specific response is under development

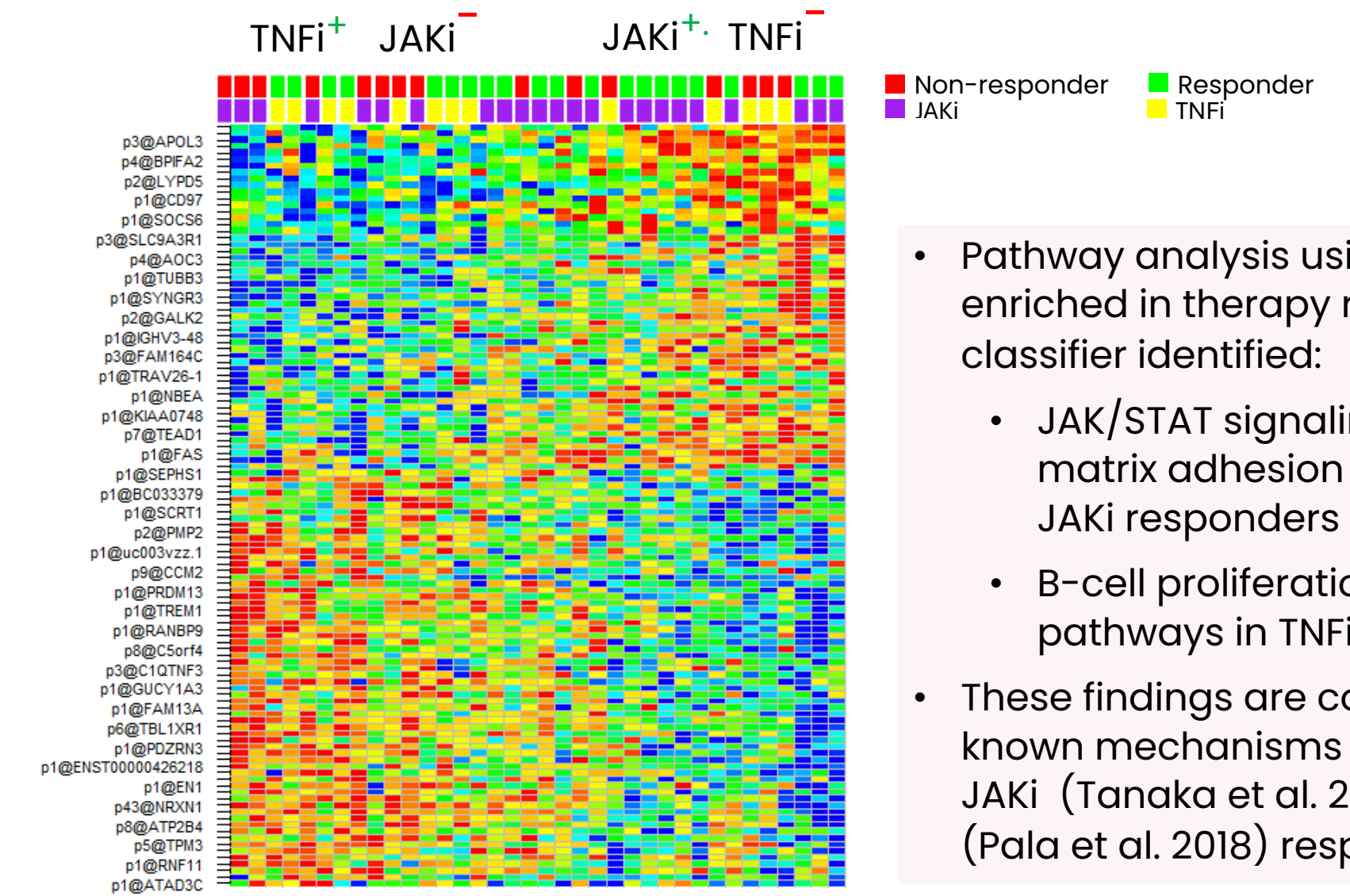

- The JAKi-response classifier demonstrated a sensitivity of 80% with a specificity of 83%.

Hierarchical clustering was performed on features identified in 2nd machine learning classifier, and pathways analysis was conducted with gene set enrichment analysis using coding genes as background.

Summary

- This proof-of-concept study establishes the use of a novel active chromatin capture method to develop a classifier capable of predicting responders from non-responders with a performance of 80% at a specificity of 85%.
- Cell enrichment and pathway analysis establish a link between the classifier features and tissue-specific RA pathobiology, providing insights into underlying cellular mechanisms driving treatment response variability in RA.
- A second machine learning classifier was also trained to predict outcomes within a single therapy class, JAKi. Using a 5-fold cross-validation framework, the JAKi-response classifier demonstrated a sensitivity of 80% with a specificity of 83%.
- Further efforts to expand the cohort will be necessary to develop a more robust classifier that captures the inherent heterogeneity of RA and delivers a positive prediction of drug-class specific therapy response.

Clinical Implications

Currently, there is a significant gap in our ability to guide therapy selection in RA. A non-invasive blood test that could predict therapy response could have significant implications for RA clinical practice and patient care.

Disclosure Information: SF is the presenter and an employee of Aqtual. DC is the lead investigator and a scientific advisory board member for Aqtual. JC is the senior author and a consultant for Aqtual.